

Control, confidence and efficiency in immittance testing





Successful immittance testing begins with you and Madsen Zodiac

MADSEN Zodiac

Madsen[®] Zodiac is an immittance testing solution from Natus, formerly Otometrics. Built from the ground up with hearing care professionals just like you, Madsen Zodiac delivers the control, confidence and efficiency you need for successful immittance testing.

Madsen Zodiac is designed to focus on the way you work, whether you are performing routine tympanograms or more advanced testing. It is reliable, responsive and easy to use. You stay focused on your patients, boost testing efficiency and increase testing success. Madsen Zodiac is suitable for use with both children and adults.



Madsen Zodiac is available in three versions:

- Quick Check
- Diagnostic
- Clinical

Each version is available as stand-alone and PC-based. Find out more on page 12.

Feel the difference. Pick up a probe and start working. It really is that easy.

Contact your local Natus representative for a demonstration.

Or, visit hearing-balance.natus.com/zodiac for more information.






An easier way to a reliable Seal

A successful immittance test depends on getting and keeping a reliable seal. However, even with the most sophisticated systems and probes, getting a seal can be an elusive and frustrating process.



There are three probes available – Quick Check for screening and Classic or Comfort diagnostic probes for more advanced testing. Each probe has a unique design for easier, faster and more stable placement. The diagnostic probes are exceptionally lightweight – they are half the weight of regular probes. This ensures quick sealing, maximum control and reliable results. EasyLock™ system for probe tips

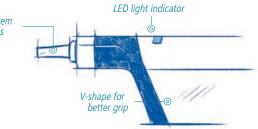

Some of our customers pick up a probe like a pen, and others like a TV remote. So, we designed our Quick Check probe to be comfortable to hold in either position.

> Mechanical Engineer from Natus

Madsen[®] Zodiac removes the guesswork in immittance testing. Ergonomic, responsive probes feel natural in your hands. You can hold them in the way that works best for you to make it easier to place the probe and feel the seal. You know immediately when to begin testing. Light indicators on the probes and shoulder pad tell you which ear you are testing, and the status of the test. Audio feedback and the probe light indicators quickly alert you if you lose the seal. And, if you lose the seal, the pump resets instantly, minimizing disruption to your test.


Focus on your patients and achieve better outcomes

Madsen[®] Zodiac makes immittance testing easier by removing the barriers that disrupt your workflow and concentration. Unique features help you stay focused on your patients and make the most of your professional skills.

Remote control buttons make it possible to stop and start tests and change ears whenever you want – without walking to and from the device. Long, sturdy cables allow you to move around freely and get closer to your patient. The comfortable no-slip shoulder strap stays put.


Dual Probe™


What's more, Madsen Zodiac is the only immittance testing solution with a dual probe option. The unique system enables you to keep the screening probe and the diagnostic probe connected at the same time all the time. Just grab the probe you need for a particular patient – without interrupting your workflow.

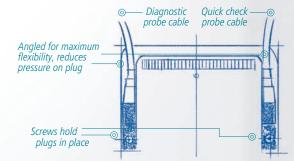

After talking with clinicians, we learned how frustrating it is to have only one probe type when you are performing all types of immittance testing. That's why we developed the Dual ProbeTM option. It's ideal for clinics that need both routine quick tymp and diagnostic capabilities.

> Audiologist from Natus

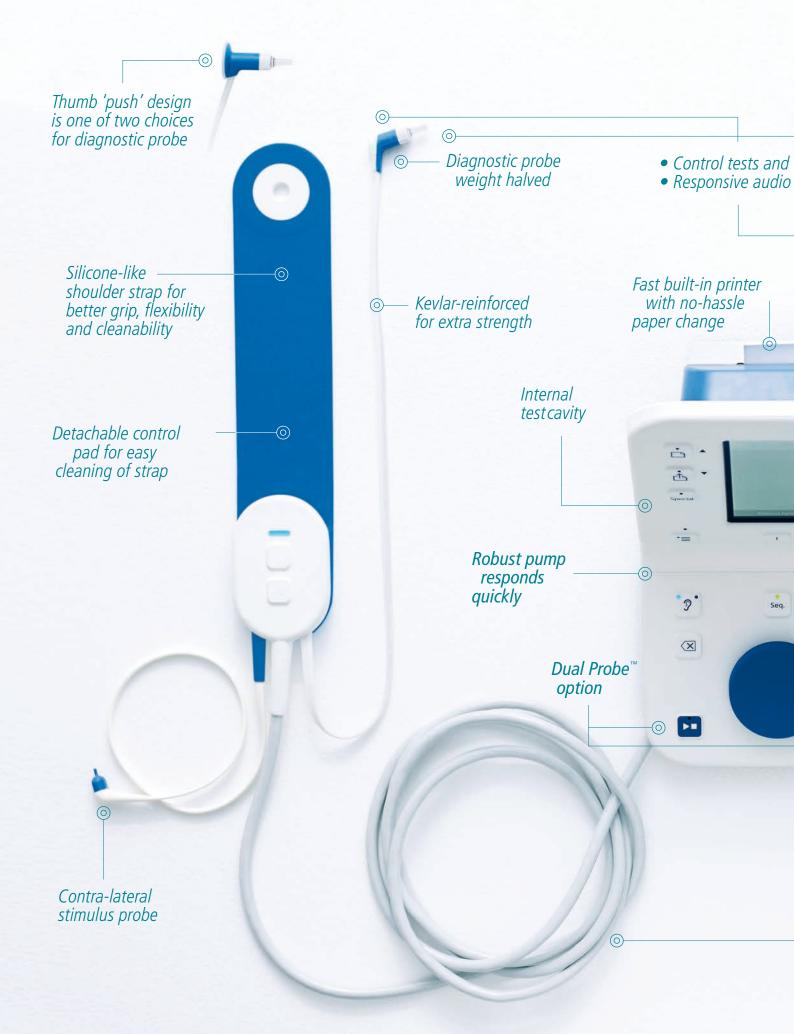

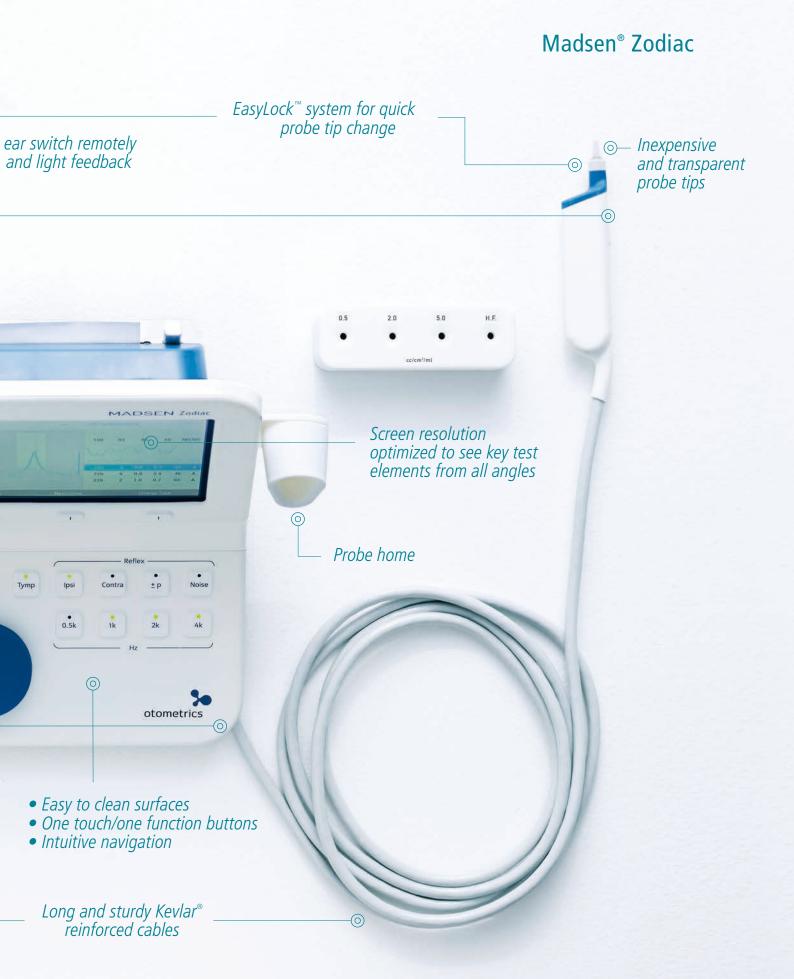

Making every step easier

Madsen[®] Zodiac streamlines immittance screening and testing. Whether you choose the PC-based or stand-alone version, Madsen Zodiac is simple to use. Just walk up and start testing – it really is that easy and intuitive.

Protocols	Quick Check Probe	Sweep Direction
	Probe Tone	Negative
 Pediatric 	Sequence	 Positive
O Protocol - 1	Auto Scale	
O Protocol - 2	Quick Check Probe	
O Protocol - 3	Pump Speed	
	Sweep Direction	
	Tymp Stop Criterion	
Apply and Exit 🔳	Screening Tests	
	Diagnostic Probe	
Save to "Adult"	Pump Speed	
	Sweep Direction	
Device Settings	Tymp Stop Criterion	
Select	Select	Select

The stand-alone version has a sleek front panel with one touch/one function buttons that give you direct access to common tests, such as tympanometry, reflex, ipsi and contra.

MANDER N. Los


A flat menu structure and self-explanatory Otosuite^{\circ} software mean you can find exactly what you need quickly – without getting lost in the system.

Seq. Tymp Ipsi Contra ± p Noise 0.5k 1k 2k 4k

The compact stand-alone solution is sturdy – yet elegant. It has a wide, color screen. The design of the 7 inch screen means you can see tests results clearly from any angle. Clear-cut display resolution and low reflection provide high visibility. The key graph elements of the curve, peak and normative area stand out on the screen – even from a distance. It was after a quick demo for an audiologist that I realized just how easy Zodiac is to use. The audiologist called a colleague over to see the system. Instead of asking me to repeat my demonstration, the audiologist was able to demo the system herself with no problem.

> Audiologist and Product Manager from Natus

Integrated LED lights in every button

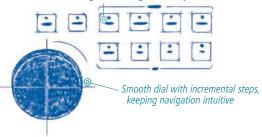




Print test results and reports quickly with the built-in printer. The paper roll is amazingly simple to change. The Otosuite software gives you the freedom to choose more printing options.

Madsen Zodiac is designed to meet the strict hygiene requirements of today's hospitals and clinics and helps to promote infection control. For example, all surfaces are easy to clean. Even the buttons are sealed, so there are no cracks where dirt and debris can accumulate. Change the inexpensive probe tips quickly – and as often as you like – with the EasyLock[™] system.



Choose the solution that works for you

Madsen[®] Zodiac is available in three versions: Quick Check, Diagnostic and Clinical.

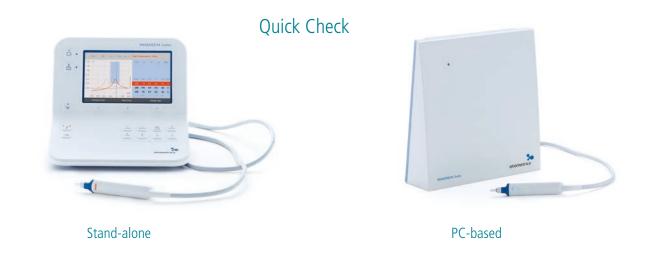



Stand-alone

PC-based

All three versions are available as stand-alone and PC-based solutions optimized for touchscreen.

Test more patients every day

Madsen Zodiac Quick Check makes tympanometry and reflex screening more efficient. A clean and simple design dedicated for screening has only the buttons and essential tests you need. Remote controls on the probe allow you to switch ears directly from the probe. Read and document the test results quickly using the built-in printer and save the results in Otosuite[®]. Remove or export as a pdf file.

Perform comprehensive immittance testing

Madsen Zodiac Diagnostic and Clinical solutions offer a comprehensive battery of immittance tests and additional tests for more extensive testing and research. Flexible setup options enable you to change to manual pressure control for advanced testing, and you can also change the settings to suit your workflow and needs. Use the dual probe feature to switch from the Diagnostic probe to the Quick Check probe at any time – without interrupting your workflow.

	Quick Check	Diagnostic	Clinical
Sweep/auto tympanometry	0	0	0
Manual tympanometry		0	0
226 Hz probe tone	0	0	0
1000 Hz probe tone		0	\odot
678 and 800 Hz probe tones			\odot
Reflex Screening	0	0	\odot
Reflex Threshold (ipsi/contra)		0	0
Reflex Decay		0	\odot
ETF-P			\odot
Admittance Recorder			\odot
B&G tympanograms			\odot

Innovative immittance testing since 1961

Natus, formerly Otometrics has driven immittance innovation for over 50 years. Poul Madsen launched the world's first commercially available electro-acoustic impedance bridge in 1961. Madsen Zodiac is our latest solution that helps our customers succeed in improving the quality of life for their patients.

Madsen Zodiac and Otosuite ...a perfect match

The Madsen[®] Zodiac workflow is based on Otosuite[®] software – the shared interface for all Natus hearing assessment and fitting solutions. Otosuite makes testing and data management more efficient, accurate and convenient.

The dedicated Otosuite immittance module is easy to learn and use. It can be customized to suit individual needs or you can standardize protocols across larger organizations. Otosuite is even optimized for touchscreen so that you can control your immittance testing from the screen of your choice.

Otosuite® Reports

With Otosuite, you can create consistent and professional reports – unique to your clinic. Otosuite Reports delivers powerful reporting capabilities and easy customization tools that enable you to design and share the reports you need quickly. Otosuite Reports combines and uses data from one or more of your Natus devices so you can create your own standardized reports. Included is concurrent report functionality that instantly fills in data as you complete each step, so it's possible to have an instant overview of your testing process and a finished report at the end of every test. And if you use Otobase[®], the Natus database, it is also possible to create standardized reports using data from all your devices - regardless of manufacturer.





OAE

Madsen[®] Astera² Clinical audiometry

Otosuite keeps everything you need for the entire patient journey in one place allowing you to work with your audiometer, OAE, video otoscope and immittance devices in one shared interface. It integrates seamlessly with Noah and external Electronic Medical Records (EMR). You can easily and safely transfer data to selected patient files, reducing errors – saving you time every day.

Feel the difference



About Natus

A leader in hearing and balance products and solutions, Otometrics was acquired in 2017, and is now known as Natus. For almost 60 years, we have been helping clinicians improve the quality of life for their clients and patients by delivering expert knowledge, reliable solutions and services and trusted partnerships. We continue to develop, manufacture and market audiological, otoneurologic and vestibular instrumentation in more than 80 countries under trusted brand names, including Madsen[®], Aurical[®], ICS[®], Otoscan[®] and Bio-logic[®]. Natus Medical Incorporated is the global brand clinicians depend on when diagnosing and treating disorders of the central nervous and sensory systems to improve patient outcomes. For more information, please visit natus.com.

Healthcare solutions with one thing in mind. You.

©2020 Natus Medical Incorporated. All Rights Reserved. All product names appearing on this document are trademarks or registered trademarks owned, licensed to, promoted or distributed by Natus Medical Incorporated, its subsidiaries or affiliates. P/N 7-26-93100-EN/05.

